

## 1 STEP

Click „View“ the study / studies:

### DICOM Library

Anonymize, Share, View DICOM files ONLINE

POWERED BY:  
Softneta

DICOM VIEWER:  
medDream

FUNDED BY:  


- Home Page
- About us
- Contact
- Library usage **1**
- ▼ Dicom
  - Transfer Syntaxes
  - SOPs
  - Modality
  - Space storage
  - About Viewer

#### View DICOM Study:

<http://localhost/?study=1.3.6.1.4.1.44316.1.54.2.20161118094029.1.6963107838193>



View



Copy Link

#### Download Anonymized DICOM Study:

<http://localhost/?requestType=WADO&studyUID=1.3.6.1.4.1.44316.1.54.2.201611180>



Download



Copy Link

#### DICOM Study MANAGEMENT Panel

<http://localhost/?manage=80d2f93567c6b12da0ba4864a0cc4bf6>



Delete

We are improving this site, so if you are missing some features or have comments, please [write us a response!](#)  
Thank you!  
DICOMLibrary Team



How to download anonymized study from Viewer?

#### What can you upload?

You can upload DICOM file or files packed as zip. Zip can contain DICOM files of studies, series and others.

#### How upload works?

Firstly test if it DICOM file or it will be skipped. Secondly try **anonymize** and then upload file to server. Same goes for zip file: try to extract and read file by file, test for DICOM format or skip, **anonymize** and upload. All extracting and anonymization is made on your browser.

#### Share?

You can share uploaded files via social networks: facebook, twitter, send by e-mail and embed DICOM viewer to forum.

#### Anonymization?


At this point - deletes information about patient and attributes to identify a person in each DICOM file. Anonymization is done in your browser.  
*Do not upload files with information written on image!*

Copyright © 2016 by DICOM Library. All rights reserved


[Terms of Service](#) | [Privacy Policy](#)

## 2 STEP

Click „Save Images“ or..

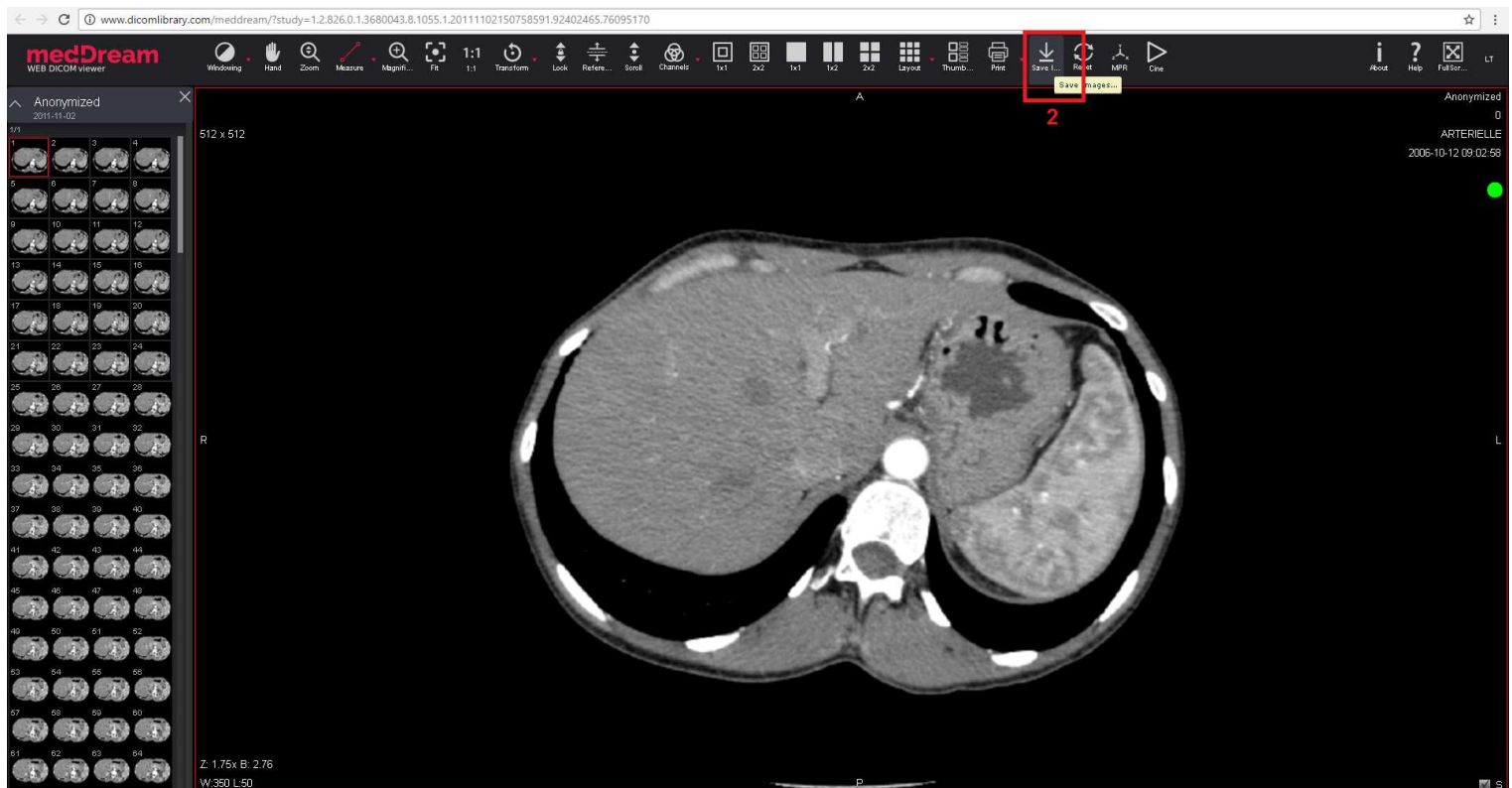

.. right mouse click and then „Save Images“:

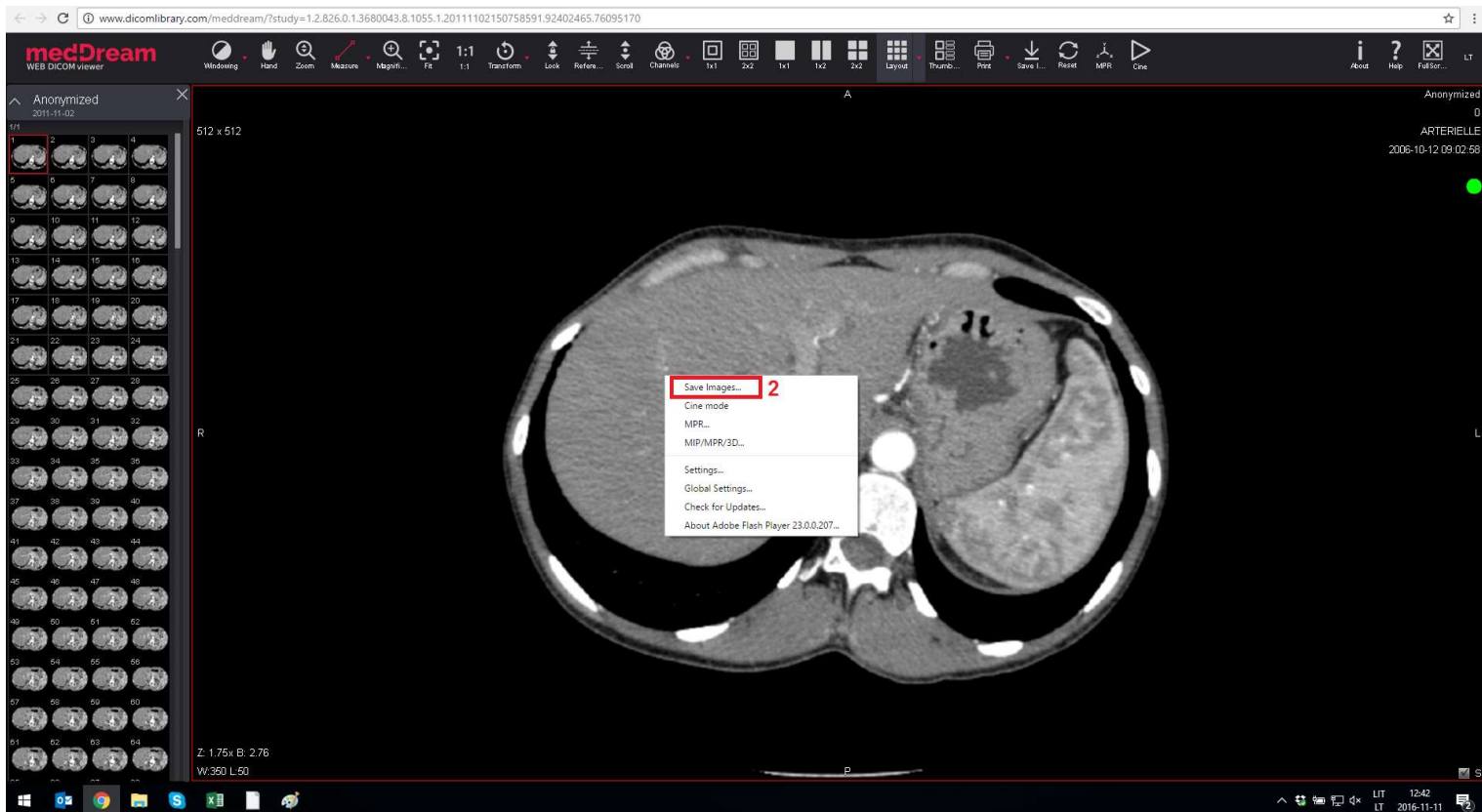

### 3 STEP

Select „DICOM“ and pick an option „Save Active Image / Video“, then click „Save“.

